

Malhotra Endonasal DCR Punch Instructions

Altomed Limited, 2 Witney Way, Boldon Business Park, Tyne & Wear, England, NE35 9PE.

For further assistance or to place orders, please contact customer services on:

Tel: (0) 191 519 0111 Fax: (0) 191 519 0283



The new Malhotra Endonasal DCR Punch has been specially designed to allow easy dismantling and processing.



This device needs to be dismantled before processing.

For full details on processing (washing and sterilizing) Altomed reusable stainless steel instruments see the Processing Instruction Information Sheet ALT I013 available from the Altomed Quality Department.

Prior to use dismantle, clean and sterilise and reassemble.

To dismantle:


1. Close the handle and then pull back the release trigger
2. Carefully slide back the moveable arm and separate from handle

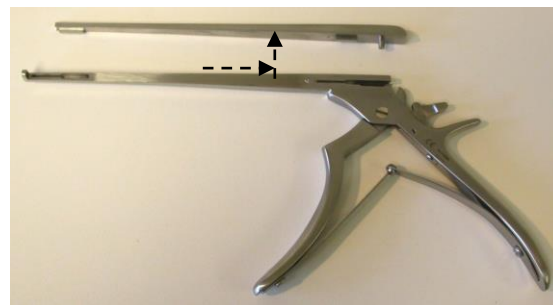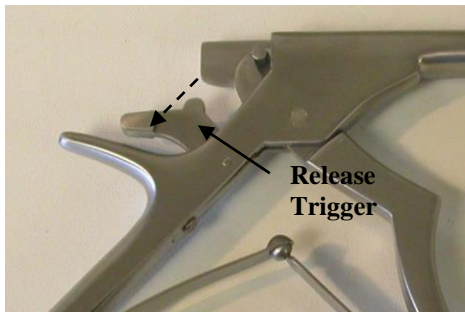


Important! Ensure the two parts of the device are kept together and are not separated from one another. Each arm is custom made for each handle.

To Reassemble:

After cleaning, lubricating and sterilising, reassemble the punch by aligning the Slip Plate with the Slip Groove and insert the Retaining Pin into the Retaining Slot. Carefully close the handle and push back the Release Trigger into the closed position.

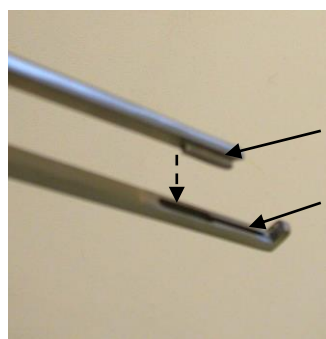

Align and insert Slip Plate with the Slip Groove

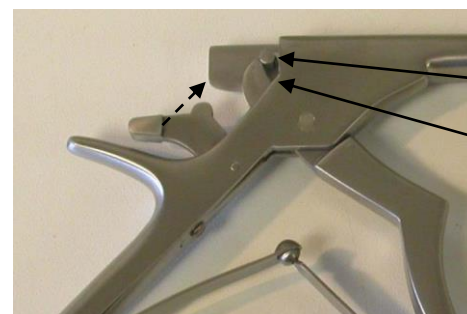

Retaining Pin fits into Retaining Slot